

## BioClean-C™ Apron -Sterile S-BCDA

## **Chemotherapy Protective Apron - Sterile**

Providing protection against a range of chemotherapy drugs, the BioClean-C Chemotherapy Protective Apron is manufactured from lightweight low-linting CleanTough blue material, and features tie tapes at the rear and an adjustable neck fastening for easy donning and comfort.

- Tested against permeation standard ASTM F739-12
- Tie tapes at rear
- Adjustable neck fastening
- Lightweight low-linting CleanTough material



### **Industries**

- Controlled and Critical Environments
- Production and Manufacturing
- Laboratory and Research







# BioClean-C™ Apron -Sterile S-BCDA

## **TECHNICAL DATA SHEET**

### **PRODUCT INFORMATION**

	S-BCDA
Material	Polyethylene/non-woven polypropylene laminate
Color	Blue
Manufacturing/QMS Audit Standards	Manufacturing QMS Audit Standards ISO 9001, PPE Regulation 2016 425 Module D
Regulatory/Standards Compliance	ASTM F739, CE 0598, ISO 11137-1:2006, EN 13034:2005 + A1:2009, EN 13934-1, EN 19935-2, EN 530, EN 6530, EN 7854, EN 863, EN 9073-4, EN ISO 13688:2013, EN ISO 14325, Category III
Storage and packaging	One piece per sealed inner PE bag; one inner bag per sealed outer PE bag; 50 outer bags per lined carton (50 pieces)
Country of Origin	China
Available sizes	S-L
Sterilization Method	GAMMA irradiation (25 kGy)
Sterilization Minimum Dose	25kGy
Sterility Assurance Level	10-6
Cleanroom Class	Class 10/ISO 4 & EU GMP Grade A
Shelf Life	Three (3) years from date of manufacture.
Construction	Adjustable neck, tie fastening at waist
Characteristics	Low particulating

## **MATERIAL PERFORMANCE TEST RESULTS**

,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,				
TEST	RESULT	PERFORMANCE CLASS		
Abrasion Resistance	10 to 100 cycles	1		
Trapezoidal Tear Resistance Cross Direction (CD)	CD 32.3 N	2		
Trapezoidal Tear Resistance Machine Direction (MD)	MD 50.6 N	3		
Tensile Strength Cross Direction (CD)	CD 47 N	1		
Tensile Strength Machine Direction (MD)	MD 94 N	2		
Puncture Resistance	10 N	1		
Repellence to Liquids – 30% H <sub>2</sub> SO <sub>4</sub>	99.0%	3		
Repellence to Liquids – 10% NaOH	98.4%	3		
Repellence to Liquids – O-Xylene	91.4%	1		
Repellence to Liquids – Butan-1-ol	93.9%	3		
Penetration by Liquids – 30% H <sub>2</sub> SO <sub>4</sub>	0%	3		
Penetration by Liquids – 10% NaOH	0%	3		
Penetration by Liquids – O-Xylene	0%	3		
Penetration by Liquids – Butan-1-ol	0%	3		
Seam Strength <sup>1</sup>	74 N	2		

<sup>1.</sup> Seam not destroyed





## BioClean-C™ Apron -Sterile S-BCDA

### **PARTICLE SHEDDING TEST RESULTS**

TEST	RESULT
Particle Shedding (Helmke Drum Test)	≥ 0.5µm (counts/min) <1700

#### **ASTM F739-12 TEST METHOD RESULTS**

DRUG	Mean Breakthrough Time (MBT), Minutes Breakthrough of the test chemical is deemed to have occurred when the permeation rate has reached 0.1 µg/cm2 /min
CISPLATIN	>240
CARMUSTINE	>240
CYCLOPHOSHAMIDE	>240
DOXORUBICINHYDROCHLORIDE	>240
5-FLUOROURACIL	>240
METHOTREXATE	>240
ETOPOSIDE	<6
PACLITAXEL	>240
THIOTEPA	55 (46,88,31)

Results achieved under controlled laboratory conditions, by accredited external testing laboratory.

#### **SIZE CHART**

S-BCDA-S: Size: S, Chest: 84-92cm (33"-36"), Height: 164-170cm (5'4"-5'6") S-BCDA-M: Size: M, Chest: 92-100cm (36"-39"), Height: 170-176cm (5'6"-5'9") S-BCDA-L: Size: L, Chest: 100-108cm (39"-42"), Height: 176-182cm (5'9"-6'0")

### **ORDERING INFORMATION**

	SIZE	S, M, L
S-BCDA	REORDER NO.	S-BCDA-S, S-BCDA-M, S-BCDA-L

#### For additional information visit us at www.ansell.com, or call us at

**Europe, Middle East & Africa Region** Ansell Healthcare Europe NV T: +32 (0) 2 528 74 00 F: +32 (0) 2 528 74 01

**Asia Pacific Region**Ansell Global Trading Center
T: +603 8310 6688

North America Region

Ansell Healthcare Products LLC US T: +1 800 800 0444 US F: +1 800 800 0445 CA T: +1-800-363-8340

Latin America & Caribbean Region
Ansell Commercial Mexico S.A. de C.V.

Ansell Commercial Mexico S.A. de C.V T: +52 442 248 1544 / 248 3133 Australia

Ansell Limited T: +61 1800 337 041 F: +61 1800 803 578

UK

Ansell Nitritex T: +44 1638 663338 Performance Standards and Regulatory Compliance

**C €** 0598





Ansell,  $^{\textcircled{6}}$  and  $^{\textcircled{m}}$  are trademarks owned by Ansell Limited or one of its affiliates. US Patented and US and non-US Patents Pending: www.ansell.com/patentmarking  $^{\textcircled{6}}$  2021 Ansell Limited. All Rights Reserved.

Neither this document nor any other statement made herein by or on behalf of Ansell should be construed as a warranty of merchantability or that any Ansell product is fit for a particular purpose. Ansell assumes no responsibility for the suitability or adequacy of an end user's selection of gloves for a specific application.

Please see product validation pack or contact Ansell customer service for specific data on use of garments with cytotoxic drugs. Garments used for protection against such drugs must be selected specifically for the type of chemicals used.

